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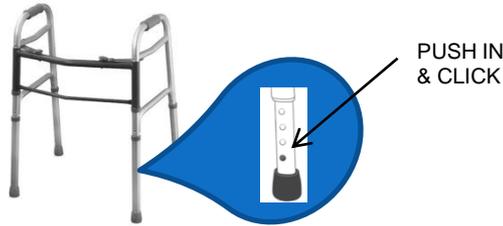
The height of the walker can be adjusted by moving the posts up or down to shorten or lengthen the leg length.

Push in the button on the post and click it up or down the necessary amount of holes in 1" increments.

Some people prefer the back posts to be positioned 1 hole or 1" lower than the front.

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With your arm extended down, handle should be at wrist height.

With hands on handgrips, elbows should be flexed at 20 to 30 degrees.

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### SAFETY PRECAUTIONS

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened.

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### LIMITED WARRANTY

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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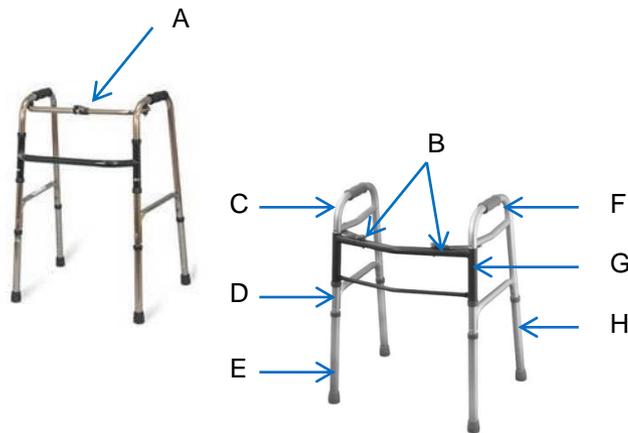
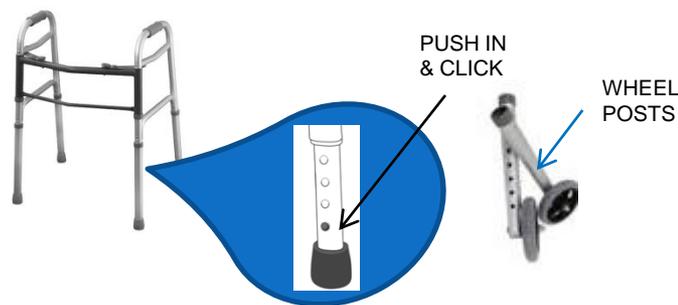
# 2 UNFOLDING AND FOLDING THE WALKER - DÉPLIER ET PLIER LA MARCHETTE

Always fully fold out the walking frame before use. To do this, pull the left side and right side away from the front of the frame. Both sides should lock in position. (1)  
To fold the walker push the single or double button mechanism and fold in the left and right side. (2)

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
Lay your walker on its side so that it is easy to access the bottom of the legs. Push the button in and slide the whole post with the rubber tips off. To install the post with wheels, slide the post over the leg and click the button until it is positioned at the correct height. Wheels should be on the outside of the frame. When space is an issue (narrow doorways) you can put the wheels facing inwards. One may want to just install wheel posts on the front (x2), leaving the rubber posts (x2) at the back OR you can replace all rubber posts with wheel posts (x4). CAUTION: the more wheels you add the faster the walker moves therefore making it more difficult to control. Never add an odd number of wheel posts (x1 or x3)

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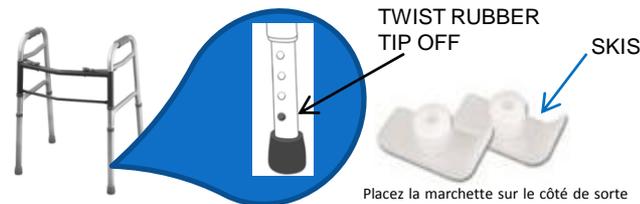Lay your walker on its side so that it is easy to access the bottom of the legs. Do not remove the posts, simply twist the rubber tips off. Once the rubber tips are removed replace with the skis. Twist skis up into the bottom of the post hole. Skis can be used on the front two posts, with rubber tips on the rear two posts OR skis can be put on the rear two posts with wheels on the front legs. Skis assist with gliding and over coming thresholds.

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