EZee Life

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Item # MFG #
CRP0926 CH2039

Limited Warranty

EZee Life brand product have a Limited Warranty for one year from the date of purchase against faulty workmanship or materials.

In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instructions set out above.

If you have a question about your **EZee Life** device or this warranty, please contact an authorized **EZee Life** dealer.

EZee Life

Installation Instructions

1. Clean Bathtub thoroughly and wipe dry before applying rail. Contact surfaces must be dry and free of soapy residue to assure secure installation.
2. Deploy knob and open until rail is wide enough to slip on side of bathtub.
3. Place Safety Rail on bathtub with knob facing outside of tub and tighten until rail feels tight and secure.
4. To test installation of rail, pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and retighten.

Safety Rail often requires retightening from time to time. Check periodically before use and retighten as needed.

CAUTION

Please use extreme care when installing on fiberglass tubs. DO NOT OVERTIGHTEN as the clamp may crack the fiberglass.

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