**4****TO INSTALL BATTERY (battery is included)**

- A. The unit uses one 1.5V (AAA) battery
- B. Turn unit so rear of monitor is facing you and pull monitor toward you.
- C. Release battery compartment door using tab at rear top of monitor and replace battery
- D. Reposition monitor on bracket on the unit

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**Safety Precautions**

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened

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**Limited Warranty**

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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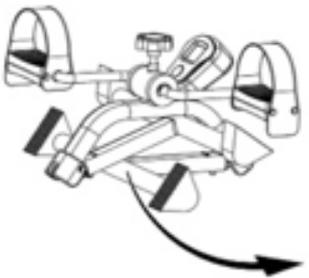
**CAUTION:** When unfolding pedal exerciser legs, take care not to pinch fingers where unit folds and locks into place.

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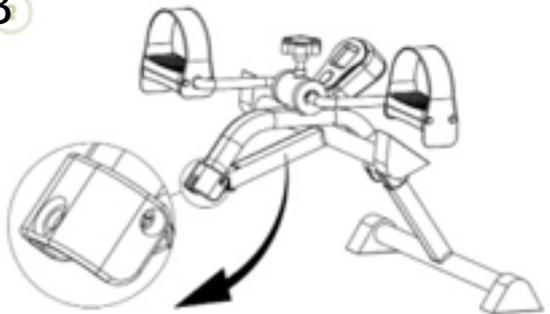
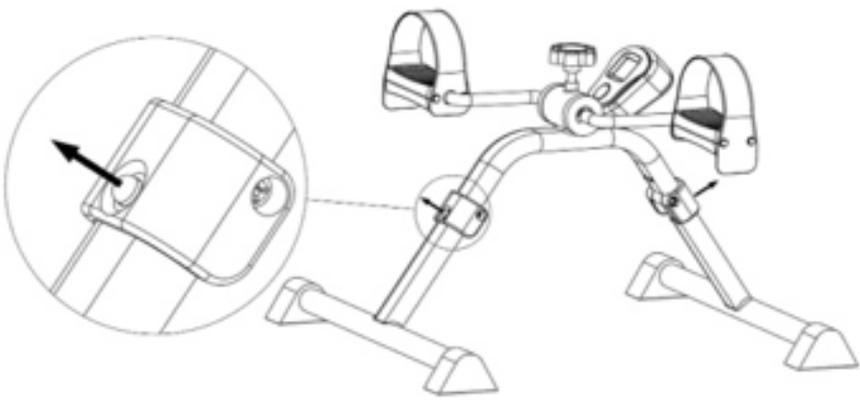
2

INSTRUCTIONS FOR USE

- A. Place pedal exerciser on the floor for leg use or on a flat surface to use the unit with your arms
- B. Sit at a comfortable distance away from the unit and begin to rotate pedals.
- C. The monitor will automatically power on when you start to exercise.
- D. When the pedal exerciser is not used for 5 seconds a "STOP" indicator in the top left corner of the window will appear and the time will also stop. The timer will resume when you continue exercising.
- E. The monitor will shut off after 5 minutes of inactivity.
- F. To reset the monitor hold the red button for 10 seconds.

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3

READOUT - YOU CAN GET DATA ON ALL OF THE FOLLOWING:

- A. Workout Time
- B. Number of Cycles from Last Reset
- C. Total Cycles for all Sessions
- D. Calories Burned from last Reset

To choose the information you want, press red button so that the arrow points to desired data or until it points to "SCAN" which will the scan through all of the data in the order above

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