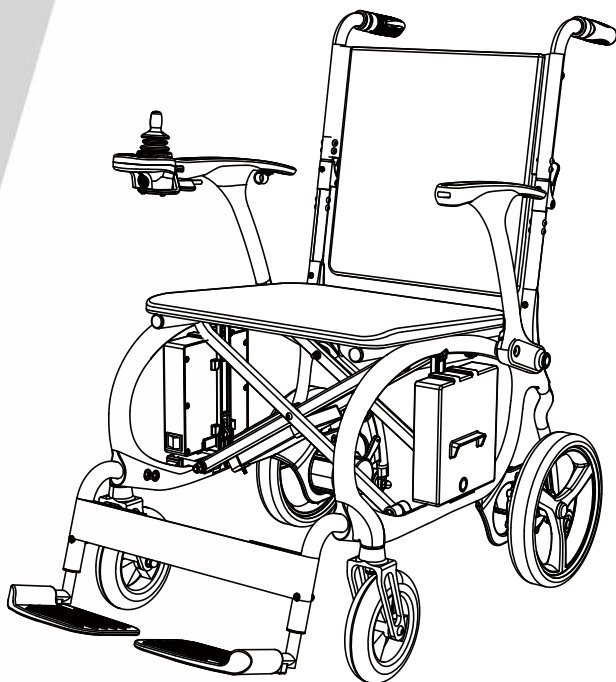
CLOUD CARBON

BY MOVO 

USER MANUAL MANUEL D'UTILISATION

ATTENTION: Please read the content of this manual before operating your Electrical Power Wheelchair

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1. USER'S GUIDELINES

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This product is distributed by Movo.

This user manual is an integral part of the product. Please ensure it accompanies every unit sold and is read thoroughly and understood before operating the power wheelchair.

This manual is based on the latest specifications and product information available at the time of publication. We reserve the right to make changes as necessary. Such changes may result in slight differences between the illustrations and explanations in this manual and the product you have purchased.

NOTE: This product and its components are not made with natural rubber latex. Consult the distributor regarding any aftermarket accessories.

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Please keep the following informations :

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Production information
(paste the barcode)

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The package contains clearly labeled accessories. Ensure all parts are present and understood. If anything is missing or unclear, contact your authorized distributor.

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- Read this user manual carefully before using your power wheelchair. Ensure you understand all components and that they are present.

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WARNING: // AVERTISSEMENT

Do not drive on roads with heavy traffic or roads that are muddy, gravelly, bumpy, narrow, snowed over, icy or canal towpaths not guarded by any fence or hedge. High levels of humidity or very cold conditions can reduce the performance of your power chair.

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Never attempt to physically modify a power chair including the addition of third-party componentry as changes to power chairs will render your warranty void and that are likely to affect conformance and risk evaluation.

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Do not suddenly shift of your body gravity as this could make the powerchair unstable. Lean forward when climbing, and lean back when going downhill, turn the speed control dial to low.

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Do not drive across a slope at an angle – always drive straight up and down the slope.

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2. SAFETY NOTICE

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! Note:

When using your power chair on streets or footpaths, local laws and regulations apply.

WARNING! Disobeying the instructions which are described in this manual may result in damage to the powerchair or body injuries to the user. Neither should it be used by people obviously suffering from physical or mental limitations that render them incapable of safely handling the power chair in traffic.

WARNING! Never exceed the maximum permissible load – see Specifications.

WARNING! Never drive the powerchair under the influence of medication or alcohol.

If you are taking medication check with your doctor or physician that your ability to control the power chair will not be impaired. Inspect all electrical connections, all controller connections, the brakes and battery charge to know the feel of your power chair and its capabilities as Pre-Ride Safety Check.

Designed solely for use by single disabled person, no persons may be transferred along with it.

WARNING! Never attempt to get onto or off your powerchair without first turning off the Joystick controller. This will prevent the powerchair from moving if accidental Joystick control contact is made.

WARNING! Never descend or climb a gradient which is greater than the recommended maximum see 4.0 Specifications.

WARNING! Never drive across (traverse) an incline, ramp etc, in any direction.

WARNING! Never use the powerchair without the anti-tip wheels correctly fitted.

WARNING! Do not use your powerchair on sand, rough terrain, or wet and slippery surfaces.

WARNING! This powerchair is not intended for use as a seat in a motor vehicle. The wheelchair could fail in a vehicle impact, leading to injury of the powerchair occupant or others.

WARNING! Do not use the powerchair if it is broken or is not functioning correctly.

WARNING! Before leaning or reaching forward, always push your bottom to the back of the chair and face the castors forwards.

WARNING! Never stand on the footplate, this will cause the powerchair to tip over and injure the user.

WARNING! Always place the powerchair on a stable, flat surface before transferring to or from the powerchair.

WARNING! The anti-tip wheels are only effective on firm ground. They will sink into soft ground such as grass, snow, or mud if the powerchair rests on them. They will lose their effect and the powerchair could tip over.

WARNING! Turning your powerchair too sharply at high speed will cause loss of control and balance and will result in the powerchair tipping over resulting in serious injury or death.

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2. SAFETY NOTICE

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2.4 INTENDED USE

This powerchair incorporates Hi-Fortune own designs and can be used in confined spaces both indoors and outdoors by the occupant and it is intended to comfortably transport adult people only excluding goods but user's weight and dimensions should be limited in 125kg(275 lbs) loading. Neither should it be used by people obviously suffering from physical or mental limitations that render them incapable of safely handling the powerchair in traffic.

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2.5 CONTRAINDICATION

The power chair is designed to transport adult people only, no goods, not any persons under the age of 16. Neither should it be used by people obviously suffering from physical nor mental limitations that render them incapable of safely handling the powerchair in traffic. These limitations could involve: hemiplegics or paraplegics, mental disorders, visual impairment.

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2.6 REMARKS ON ELECTROMAGNETIC COMPATIBILITY (EMC)

We wish to point out that sources of electromagnetic waves (e.g. cell phones) are liable to create interferences. The electronics of the power chair itself could affect other electric appliances too. To reduce the effect of electromagnetic sources of interference, please read the following warnings:

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Interfering electromagnetic fields may have a negative effect on the power chair's electronic systems. These can include: Disengagement of the engine brake; Uncontrollable behavior by the power chair; Unintentional steering movements. In the presence of very strong or enduring interfering fields, the electronic systems could even break down entirely or suffer permanent damage.

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2. SAFETY NOTICE

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POSSIBLE SOURCES OF RADIATION INCLUDE:

Portable receiver and transmitter installations (receiver and transmitter with fixed antenna mounted directly on the transmitting unit)

- Transmitting and receiving sets
- Cell phones or "walkie talkie"
- portable TV, radio and navigation devices
- other personal transmitting devices
- Mobile medium-range transmitting and receiving devices (e.g. car antennas)
- Fixed transmitting and receiving sets
- Fixed mobile transmitting and receiving devices
- Fixed radio, TV and navigation systems
- Long-range transmitting and receiving devices
- Radio and television towers
- radio sets

Other home devices like CD player, notebook, cordless phones, AM/FM radios, electric shavers and hair dryers, so far as we know, will have no influence provided that they function perfectly and their cabling be in an excellent condition. Please conform to the operating instructions accompanying such electric instruments to ensure trouble free operating of your power chair.

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3.1.1 This product is composed of frame, wheel, seat cushion, armrest, lithium battery, motor and controller. Lightweight and compact design; The whole vehicle can be folded, and it can be easily pulled after folding; The handrail can be turned backward, convenient for the elderly to move; Configure a high-performance controller to allow the elderly to operate flexibly; The battery case can be removed for easy charging.

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3.1.2 The waterproof grade of this product is IPX4, and the application part of the shock protection degree is type B.

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3.1.3 This product is designed to simplify operation and facilitate user use.

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3.1.4 Ergonomic seat design provides adequate comfort for the passenger.

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3.1.5 This product is suitable for use indoors and on flat roads near buildings.

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3.2 MAIN STRUCTURE // STRUCTURE PRINCIPALE

The wheelchair consists of carbon fiber frame, seat, carbon fiber armrests, battery and controller.

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
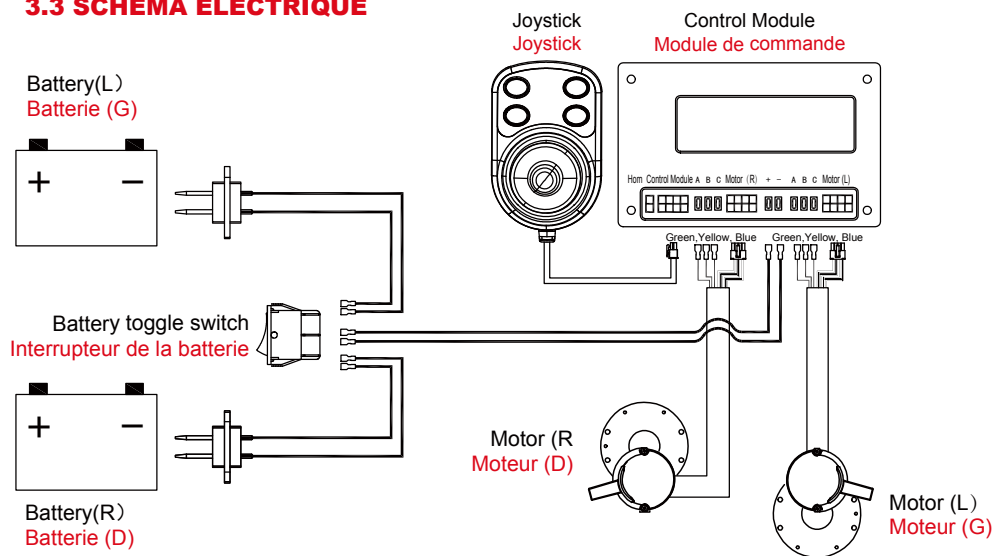
Insert the upper bracket of the controller into the fixed tube of the armrest as shown in the diagram and then tighten it with the knob. Install oppositely if operated with left hand.

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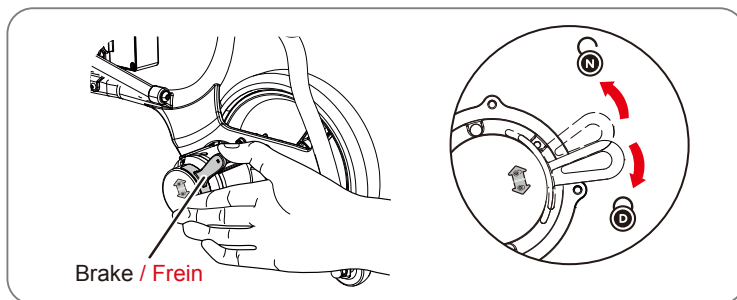4.2 Brake // Frein

Pull the handle to the open position (N) to put the wheelchair in the brake-release position and you can push the wheelchair manually. When the handle is pushed to the closed position, the wheelchair is in the braking state and cannot be pushed.

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It is forbidden to use the opening (N) function when the wheelchair goes uphill or downhill in order to ensure the safety of the user and wheelchair.

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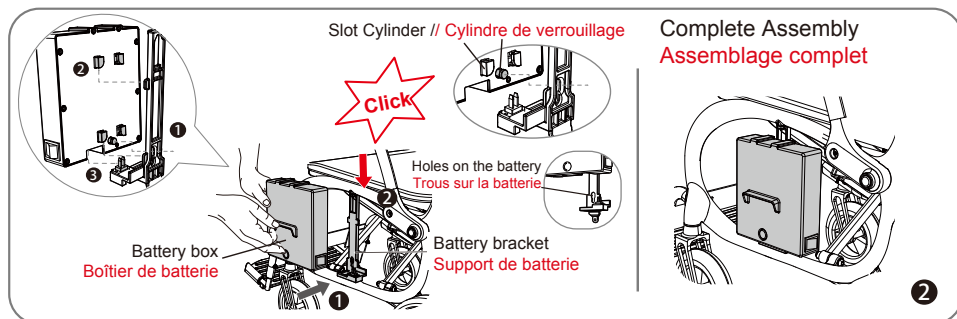
4.3.1 Assembly // Assemblage :

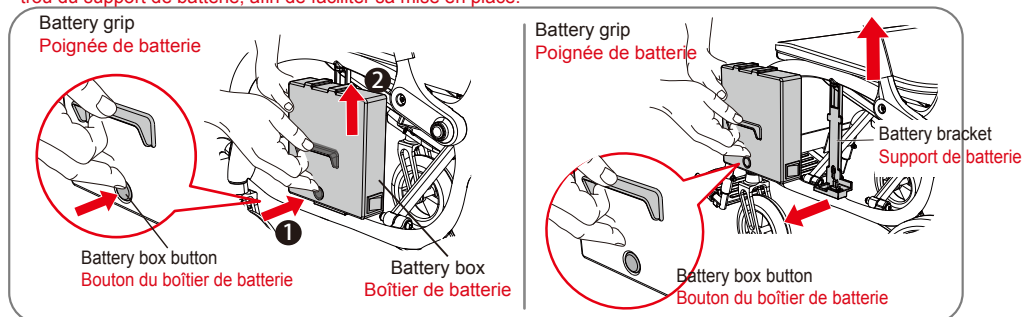
Attach the battery boxes on both sides of the wheelchair. Four slots and a cylinder behind the battery box are requested to align with the holes and fulcrums on the battery bracket and then clamp downwards until a "click" sound is heard and the battery boxes are properly attached.

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Attention:

Before the battery assembling, make sure to align the cylinder behind the battery box with the hole on the battery bracket, which can easily get the battery attached in place.

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Press down the button on the battery box and lift the battery grip up to remove it.

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Attention:

When lifting the battery, you can let your hand that presses the battery box button go.

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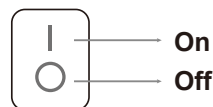
Attention:

① Make sure the switch is on position "I", that is, the power switches on.

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
② Hold the battery box with both hands to prevent it from falling off when assembly and disassembly of the battery box.

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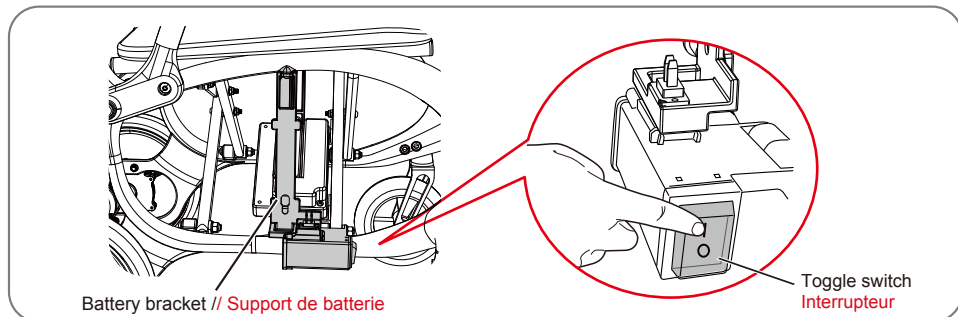4.4 Battery Toggle switch

A toggle switch is equipped onto the right battery bracket. Toggle the left and right battery with toggle switch. Position "I" and "O" (see Figure on the right) means the left or right battery is respectively activated.

NOTE: Left and right directions are determined based on the forward direction of the wheelchair user.

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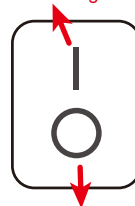
Caution

Always turn off the power from joystick before switching the left/right batteries.

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
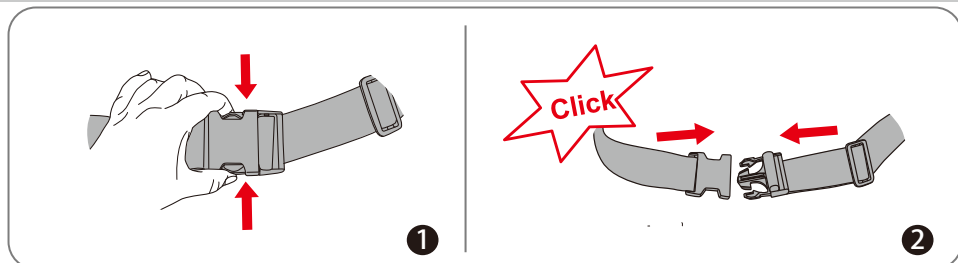
⚠ WARNING / AVERTISSEMENT :

Always fasten the seat belt when seated in the wheelchair.

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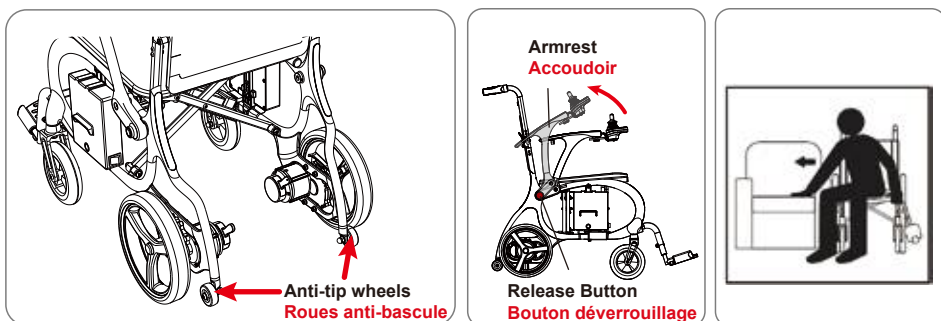4.6 Anti-tip Wheel & Armrest // Roues Anti-Bascule & Accoudoir

4.6.1 The wheelchair is equipped with anti-tip wheels designed to help prevent the wheelchair from tipping over in emergency situations. The anti-tip wheels are not removable.

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WARNING / AVERTISSEMENT :

Do not drive the wheelchair after the anti-tip wheels are removed or damaged.

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WARNING / AVERTISSEMENT :

Anti-tip wheels are only suitable for hard surfaces. If the road is muddy, the anti-tip wheel may get stuck.

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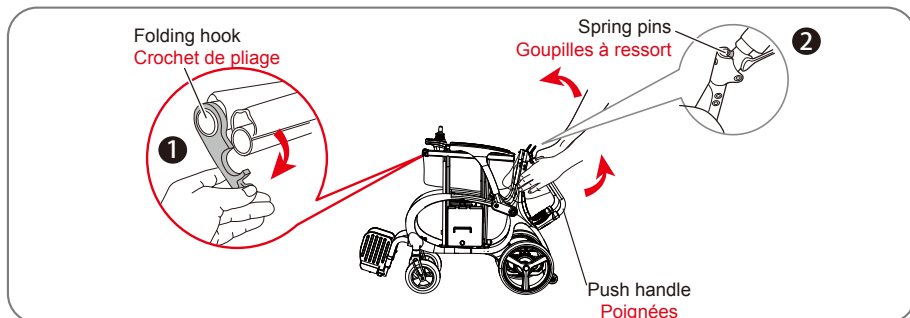
mevo

4.7 Folding and Unfolding

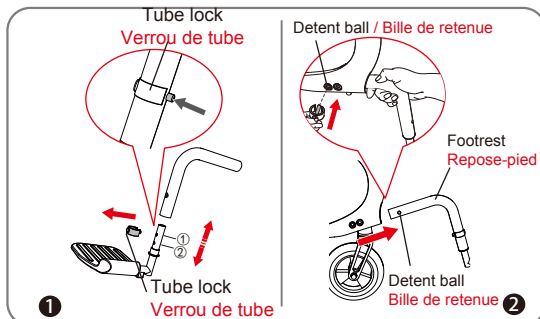
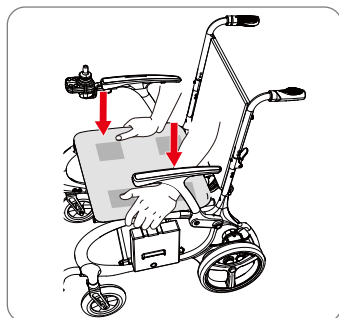
4.7.1 Folding: Remove the wheelchair from its carton and unlock the folding hook (see Fig. 1). Then, hold the handles as shown in Figure 2 and lift them until the spring pins engage with the push handles.

4.7.1.2 Unfolding: Press down on the seat crossbars with both hands until the wheelchair is fully unfolded.

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4.7.2 Footrest Removal and Height Adjustment

1. Height Adjustment:

Press the protruding button and remove the tube locking pin. The footrest height can then be adjusted up or down by aligning with the mounting holes (see Figure 1).

2. Footrest Removal:

Hold the footrest tube with one hand. With the other hand, press the ball bearing inward by pushing the pin using the front end of the tube locking pin. At the same time, pull the footrest tube outward to remove it (see Figure 2).

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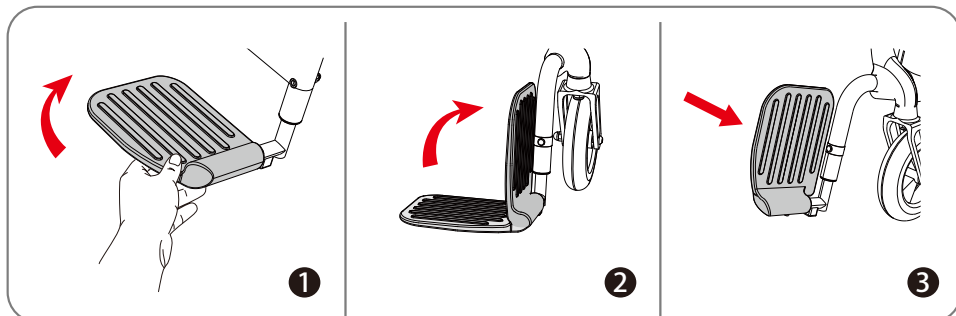
4.7.3 Folding and Unfolding the Footrest (See Figures)

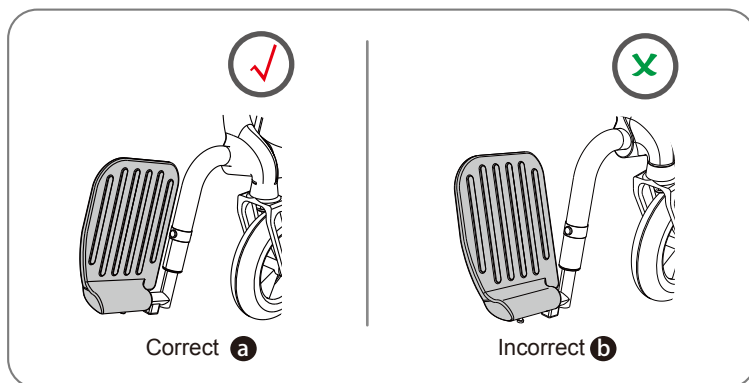
To fold the footrest, lift it upward (Fig. 1) until it is fully folded (Figs. 2 and 3).

To unfold the footrest, reverse the above steps.

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Note:

Correct Folding (See Figure A): Flip the footrest upward and ensure it is fully folded and securely locked in place.

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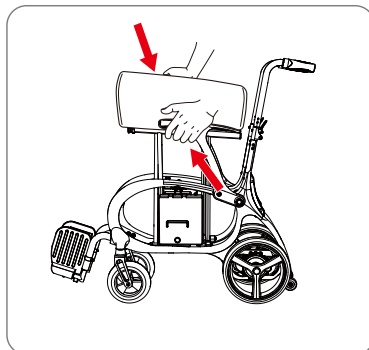
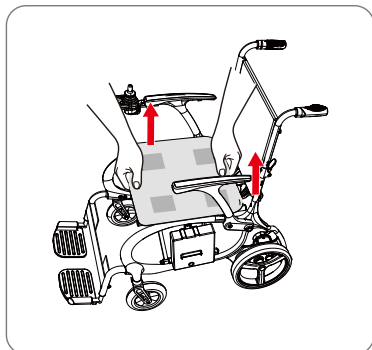
mevo

4.8 Folding

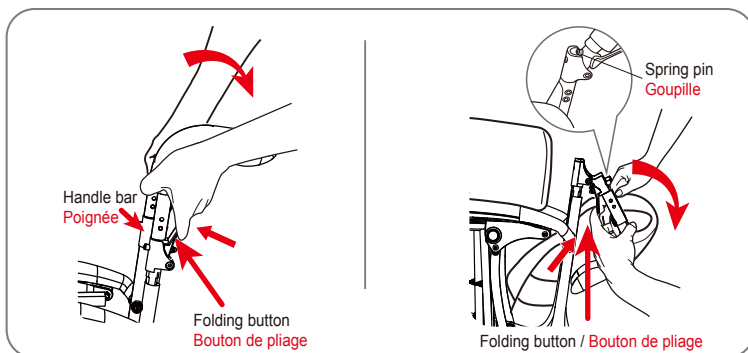
4.8.1 Footrest Folding: Fold the footrest by flipping it upward (see Section 4.7.3).

4.8.2 Wheelchair Folding: Grasp the seat upholstery as shown in the image and pull it upward with both hands to fold the frame.

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
4.8.2.2 Hold both armrests and push inward simultaneously until the wheelchair is fully folded.

4.8.2.3 Hold the handlebar and press the folding button. At the same time, lift the handlebar slightly until the spring pin disengages from the hole in the handlebar sleeve, then fold it down.

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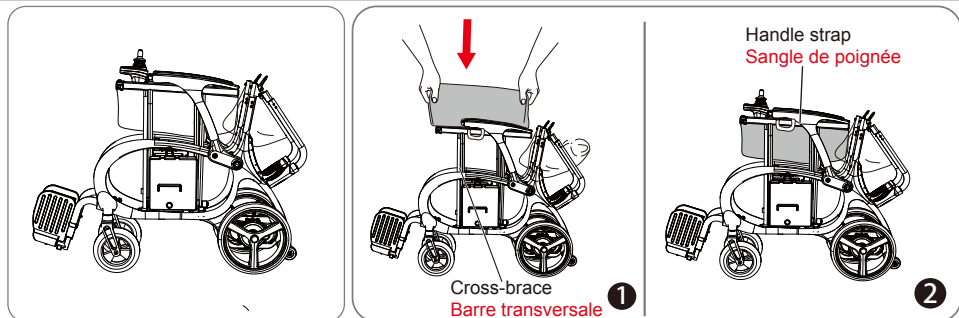
4.9 Storage

After folding, the wheelchair can be stored in an upright position to save space and allow for convenient storage.

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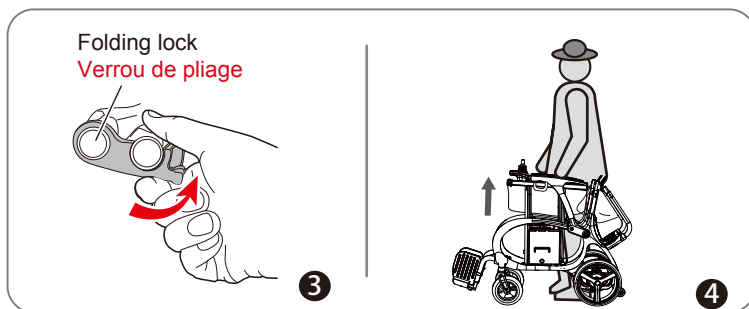4.10 Portability

1. Remove the seat cushion and fold down the seat upholstery while folding the wheelchair (Figure ①), exposing the handle straps on both sides of the seat (Figure ②).
2. After folding the wheelchair, fasten the folding lock to prevent the frame from unfolding (Figure ③).
3. When lifting or carrying the wheelchair, use the handle straps to easily lift the entire unit (Figure ④).

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5.5 DRIVING ON INCLINES, SLOPES AND RAMPS

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6.1 GENERAL INFORMATION & INSTRUCTIONS ON CHARGING

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Your power chair is equipped with a sealed, maintenance-free lithium battery. The battery is recharged using the supplied external charging system.

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- Fully charge the battery for at least 10 to 12 hours before using the power chair for the first time. The battery will reach full capacity after approximately 10 to 20 charge cycles (break-in period). This period is necessary to ensure optimal performance and longevity.

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- Keep the battery fully charged to ensure optimal performance. Recharge the battery daily after each use, even if it has only been partially discharged. Depending on the level of discharge, a full recharge may take up to 12 hours.

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- If the battery indicator reaches the red zone, charge the battery for a minimum of 16 hours, even if the charger indicates that charging is complete.

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- Even if the power chair is not used for an extended period, the battery should be charged regularly to maintain a full charge.

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- Avoid operating the battery at a low charge level without recharging it fully on a regular basis.

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- Do not charge the battery in extreme temperatures. Charging is not recommended at temperatures above 40 °C (104 °F) or below 10 °C (50 °F).

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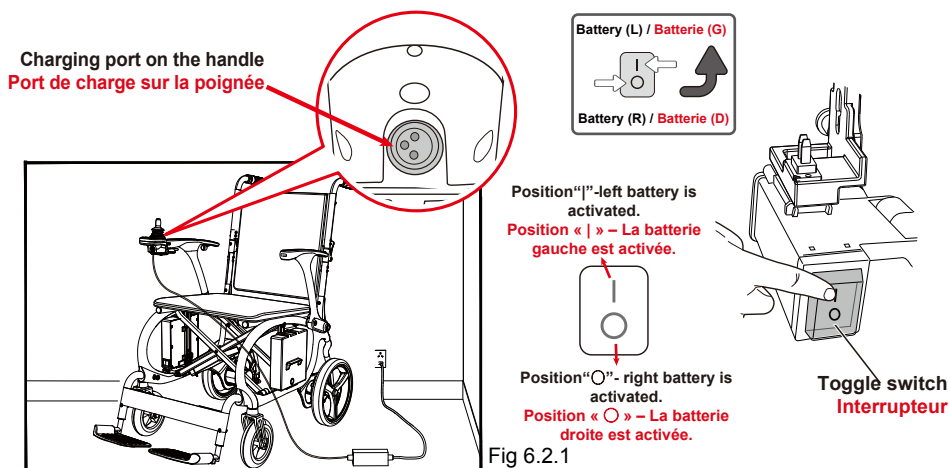
- Protect the charger from heat sources such as heaters and direct sunlight. If the charger overheats, the charging current will be reduced and the charging process will be delayed.

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

Note: Only one battery can be charged at a time. Set the toggle switch to "I" to charge the left battery, or to "O" to charge the right battery. Left and right are determined based on the user's forward-facing direction.

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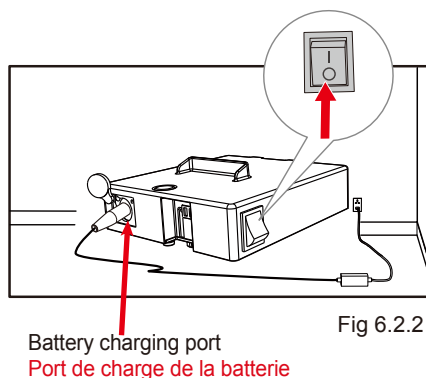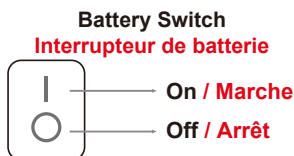Note: The battery box switch must be set to "I" during charging (power on).

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6. BATTERY AND CHARGER

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WARNING: Turn on the battery box switch before charging.

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WARNING: Using an incorrect charger may cause battery damage or explosion. Use only the supplied charger or one approved by Green Pine.

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WARNING: Do not use or store the battery near heat sources such as fires or heaters.

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WARNING: Do not open or disassemble the charger. If it is not working properly, contact your authorized provider.

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WARNING: Do not use the battery if it emits an odour, becomes hot, or appears abnormal. Stop use immediately and remove it from the device or charger.

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WARNING: Risk of electric shock if the charger gets wet. Keep it dry at all times and charge only in a dry environment.

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Protect your power chair with regular inspections. We recommend having it checked at least once a year by a qualified dealer using approved parts. Regular maintenance helps ensure reliable performance, extends lifespan, and improves ease of use.

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Your power chair has a unique identification number located on the chassis and battery box. This number allows your authorized dealer to verify the product's age and purchase date with MOVO EVOLUTION INC. if needed.

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7.9 END OF USE // FIN D'UTILISATION

This symbol indicates that the product must not be disposed of with household waste. Follow local laws and regulations for proper disposal. At the end of its service life, take the wheelchair to a designated recycling facility. Proper disposal helps conserve natural resources and protects the environment.

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Each component must be recycled by an authorized facility in accordance with local laws and regulations. Plastic parts, frame, and tubing can be taken to specialized recycling centers at the end of use.

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LIMITED WARRANTY – IMPORTANT NOTICE: This warranty must be completed and returned to MOVO EVOLUTION INC. within 14 days of purchase to be valid.

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8.1 ONE-YEAR LIMITED WARRANTY // GARANTIE LIMITÉE D'UN AN

For one (1) year from the date of delivery, we will, at our discretion, repair or replace any defective part or electronic component free of charge for the original purchaser, following inspection by an authorized Movo representative.

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Note: Increased motor or transaxle noise is not covered under warranty. This is typically caused by excessive strain on the power chair.

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8.2 BATTERIES // BATTERIES

Batteries are covered by a six (6) month warranty from the original manufacturer, with an additional six (6) months at the discretion of Movo Evolution inc. **Note: Gradual performance loss due to prolonged discharge, exposure to cold conditions, or heavy use is not covered.**

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Exclusions also include damage caused by:

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Note: Gradual performance loss due to prolonged discharge, exposure to cold, or heavy use is not covered under warranty.

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This warranty applies only to the original purchaser. Proof of purchase (original receipt) is required for any warranty service. This warranty covers defects in materials or workmanship under normal use. It does not cover damage during shipping, or failures caused by non-approved parts, accidents, misuse, abuse, neglect, improper handling, modification, commercial use, unauthorized repairs, or events beyond the control of Movo Evolution inc.

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There are no other express warranties. To the extent permitted by law, any implied warranties (including merchantability or fitness for a particular purpose) are limited to:

1. One (1) year from the date of purchase.
2. Repair or replacement of defective parts only.

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Note: Register your warranty with your dealer within one month of purchase. This card is valid only if stamped by the dealer or manufacturer and must be presented for warranty service.

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Buzzer Self-Diagnosis: The number of buzzer beeps indicates the type of fault.

Refer to the following checklist to troubleshoot the issue. If the problem persists, contact your authorized dealer.

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Alarm example: If two fast beeps followed by five slow beeps , which stand for the buzzer alarm number will be 2+5, and the fault may be caused by the left brake or connector is in poor contact.

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9.1 Suggest to change the battery once a year from our dealer or agency.

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9.2 Suggest to change the tyre by contacting our agency when its pattern is worn to half.

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