EZee Life

EZEE Travel LITE



USER'S MANUAL

Table of contents:

	Page
Introduction	2
Safety	3
System Components	4
Folding Instructions	6
Operating your <i>EzeeTravel Lite</i>	8
Moving and Transporting your EzeeTravel Lite	12
Helpful Hints for Everyday Use	13
General Maintenance and Inspection	14
EMI – Electromagnetic Interference	16
Fault Finding	19
Disposal and Recycling	20
Technical Data	20
	Safety System Components Folding Instructions Operating your EzeeTravel Lite Moving and Transporting your EzeeTravel Lite Helpful Hints for Everyday Use General Maintenance and Inspection EMI – Electromagnetic Interference Fault Finding Disposal and Recycling

NOTE: Design details may change without notice.

INTRODUCTION

The *EzeeTravel Lite* is intended for use by individuals that are able to walk, but suffer from mild mobility limitations.

The user must have sufficient arm and leg strength to get on and off the *Ezee Travel Lite* alone and to safely steer under all driving conditions.

The **EzeeTravel Lite** is intended for indoor and restricted outdoor use on pavements or paved footpaths only during daylight hours.

The *EzeeTravel Lite* must not be used as a transportation vehicle on roads or freeways used by cars.

The **EzeeTravel Lite** does not have lights and reflectors and therefore must not be used in the dark or in limited visibility conditions.

CAUTION: Failure to follow the instructions contained in this manual may result in injury to the user or to other persons.

2. SAFETY

During your initial use of the *EzeeTravel Lite* we recommend caution as you practice operating the unit in various situations. Keep the speed at a reduced level until you are comfortable controlling the scooter. Follow the safety tips and instructions and you will be comfortable maneuvering through doorways, on and off lifts, over moderate terrain and up and down ramps.

These safety considerations and tips will help you to operate the **Ezee Travel Lite** safely. The **Ezee Travel Lite** is a powerful electric vehicle; these rules will help you prevent personal injury and damage to your scooter.

- 1. Read this manual and all labels before operating.
- 2. Do not carry passengers or exceed the maximum weight capacity.
- Do not mount or dismount the Ezee Travel Lite unless it has come to a full stop and is turned off.
- 4. Do not back up onto uneven inclines or surfaces.
- 5. Always switch the speed selector to low when driving in a confined space.
- 6. Always reduce speed when turning.
- 7. Do not operate your scooter when under the influence of alcohol, medications or drugs that may impair your safety.
- 8. Always keep your feet on the foot platform when driving.
- 9. Do not sit on the scooter while being transported in a moving vehicle. Always fold down and secure your *Ezee Travel Lite* and transfer yourself to a vehicle seat.

When using a mobility scooter you need to consider the following:

Common sense

- Slow down for curves
- Balance your weight for stability by not leaning to one side
- Do gradual turns not sharp or fast turns
- · Do not drive in hazardous conditions
- Drive at speeds reasonable for the terrain and conditions
- Do not drive if physically unable to operate your scooter safely

Cognitive state

- User needs to be aware of their speed
- User needs to understand forward and backward driving
- User has to understand the environment in which the scooter is being used
- User has to obey and understand the traffic rules
- Scooters are to be used on sidewalks, if there are no sidewalks, users must behave as pedestrians.
- Be aware of potential hazards, on the road (pedestrians, automobiles, weather elements, terrain, etc.)

Motor skills

- User has to have good up trunk support and posture control
- User has to have good dexterity in fingers and hands to operate controls
- User has to have good balance
- User has to have the ability to get on / off scooter unaided
- User has to have good eye sight

1. SYSTEM COMPONENTS AND DETAILS

1.1. *Travel Lite Compact* parts – (Figure 1)

- 1. Battery pack (detachable)
- Front column (foldable)
- 3. Column lock/release triggers
- 4. Controller cover
- 5. Charging socket
- 6. Front drive wheels7. Foot platform
- 8. Seat shell
- 9. Seat cushions

- 10. Rear wheel
- 11. Utility basket (holding capacity 20lbs/9kg)
- 12. Control Panel
- 13. Freewheel switch
- 14. Transaxle motor
- 15. Freewheel Handle (Fig. 1A)
- 16. Charger and connecting cables (Fig. 1B)



Figure 1

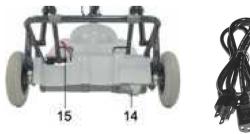




Figure 1A

Figure 1B

1.2. Control Panel – (Figure 2)

- 1. On-Off Switch
- 2. Hand control lever (right and left)
- 3. Speed adjusting knob
- 4. Indicator light (LED)
- 5. Battery Gauge
- 6. Hand-grips



BELL: If the user requires an audible warning device, a standard bicycle bell can be attached to the handlebar.

2. FOLDING INSTRUCTIONS

2.1. Unfolding your EzeeTravel Lite

- Pull the Column Lock Triggers (Fig. 1/3) and lift Steering Column from Seat (Figure 3)
- 2. Lift the Seat Backrest upright
- 3. Grasp top of Backrest and pull upwards to unfold the Seat Frame and let the Crossbar (Fig. 4-A) fall into the Crossbar Supports (Fig. 4-B)



Figure 3

2.2. Folding your Ezee Travel Lite

- 1. Grasp top of Backrest, lift it a little bit just to release the Crossbar (Fig. 4-A) from the Supports (Fig. 4-B) and fold the Seat Frame down
- 2. Fold Seat Backrest down
- 3. Pull Column Lock Triggers (Fig. 1/3) and fold Column down over Seat (Figure 3)



Figure 4

2.3. Front Column Adjustment (Figure 5)

To adjust the angle of the front column, pull the column lock triggers upward. Move the column into the desired position and release. Push lightly back and forth on the column to ensure that it is locked in the desired position.

- 1. For driving the Ezee Travel Lite, adjust column to one of the two rear-most positions (Fig. 5/1, 2).
- 2. For getting in and out of the Ezee Travel Lite seat comfortably, the column may be moved away from the seat to the forward-most position (Fig. 5/3).

CAUTION: When getting in and out of the Ezee Travel Lite seat, step only on CENTER of foot platform. DO NOT STEP ON SIDE OF FOOTPLATFORM to avoid tipping the Ezee Travel Lite.



1. Operating (driving) position



2. Operating (driving) position II



Figure 5

3. OPERATING YOUR EzeeTravel Lite

3.1. Control Panel operation features

The switch located on the right side of the handle control cover (Fig. 6/1), will turn the power of the *EZee Travel Lite* ON or OFF.

NOTE: Turn your *Ezee Travel Lite* switch to the OFF position when not in use in order to conserve battery power, and prevent unintentional operation.

The top speed may be adjusted by turning the Speed-adjusting knob (fig 6/3). By turning the knob to the HI position, your *Ezee Travel Lite's* top speed will be up to 6kph/3.7mph. When the knob is set to the Lo position, the speed will be reduced to a slow crawl, ideal for less experienced drivers and for operating



Figure 6

3.2. Indicator Light (Fig. 6/4)

- □ *Indicator light steady* This indicates that all is well.
- □ *Indicator light flashes slowly* The controller is functioning correctly but batteries should be charged as soon as possible. The Battery Gauge (Fig. 6/5) shows the charge that is left

- □ *Indicator light flashes* This indicates a possible fault. Please follow the following procedure:
 - 1. Switch off the **Ezee Travel Lite**
 - 2. Make sure that the *Ezee Travel Lite* is not in the FREEWHEEL position.
 - 3. Make sure that the charger is not connected to the *Ezee Travel Lite*.
 - 4. Make sure that the control lever is in the middle position.

CAUTION: If control lever does not return by itself to the middle position - do not operate. Contact your service agent.

5. Switch the *Ezee Travel Lite* on again and the indicator light. If it flashes rapidly again, switch off and do not operate. Contact your service agent!

3.3. Battery Gauge (Fig. 6/5)

The Battery Gauge (Fig. 6/5) shows the battery charge that is left.

3.4. Driving your Ezee Travel Lite

To drive forward, pull the right side of the hand control lever (Fig. 2/2) towards you. You can also move forward by pushing the left side of the control lever away from you. By releasing the control lever, you will gradually come to a stop, and the *EzeeTravel Lite* brake will be applied automatically. To operate in reverse, pull the left side of the hand control lever towards you.

3.5. Safety considerations when driving your EzeeTravel Lite

Footpaths and sidewalks appear to be level but are usually slightly sloped for drainage of rainwater. Counter-
steering may be necessary for overcoming excessive slopes. In such cases the speed should be reduced as
necessary.

☐ When approaching ditches, bumps or similar obstacles – reduce speed as necessary.

□ Do not operate *Ezee Travel Lite* in places or on surfaces where a loss of wheel grip could be hazardous, for example on wet grassy slopes.

3.6. Armrests (optional)

To enable easier entry to and exit from your *Ezee Travel* the armrest can be lifted, see Fig. 10A.



Figure 10 A

Battery information

3.7. Batteries

The EzeeTravel Lite is designed to use sealed lead acid, maintenance-free, 12V batteries.

Always turn your *EzeeTravel Lite* OFF before removing and installing batteries. To get the maximum out of your batteries, it is recommended to fully charge the batteries overnight after every day of use. **Reconnecting the charger when the batteries are partially discharged or fully charged will not harm them.** Extended charging is good for the batteries and you cannot overcharge them.

Battery range depends on operator weight, terrain and the condition of batteries used. Charge your batteries daily and/or overnight when the *Ezee Travel Lite* is not in use.

Battery Recharging Instructions

To recharge, turn off *EzeeTravel Lite*. It is possible to either charge the battery while connected on the *Ezee Travel Lite* (see instruction 1 below) or directly (instruction 2).

- 1. Plug the charging connector into the **Ezee Travel Lite** charging socket and then connect the charger to an electrical outlet socket.
- 2. Plug the charging connector in the back of the battery pack and then connect the charger to an electrical outlet socket.

The indicator light is red when the power is on and the charger is not connected to the battery. During charging, the indicator light on the charger will be orange. When the battery is approaching full charge the indicator light will turn green. In order to achieve the maximum battery capacity and efficiency, it is advised to charge them overnight, and at least four hours after the indicator light has turned to green.

Safety Instructions

CAUTION: Failure to follow these instructions may result in personal injury or property damage.


- 1. Never smoke or allow an open flame in the vicinity of the batteries.
- 2. Use the charger for charging lead acid batteries only. It is not intended to supply power to an extra-low-voltage electrical system or to charge dry cell batteries.
- 3. Never charge a frozen battery.
- 4. Do not operate the charger in a closed-in area or restrict ventilation.
- 5. Prevent external damage to AC and DC cords. Do not use charger if cords are damaged.
- 6. Connect DC charging cord to the Charging Socket before connecting charger AC supply cord to the electrical outlet, and disconnect the AC cord from the electrical outlet before disconnecting charge cord.
- 7. Be extra cautious not to drop a metal object onto the battery case.
- 8. Remove jewelry.

Battery Storage

Before storage, battery packs should be fully charged, and should be recharged at least once every three months.

Store batteries indoors in a dry environment.

Storage temperature range: -20°C (-4°F) to 40°C (104°F) Charging temperature range: 0°C (32°F) to 40°C (104°F) Operating temperature range: -20°C (-4°F) to 50°C (122°F)

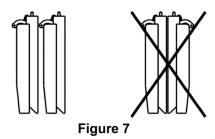

Battery Replacement and Disposal

The *EzeeTravel Lite* battery pack contains two 12V maintenance-free batteries. Replacement of the batteries is only permitted to an authorized dealer.

Battery cells may emit minor acid fumes.

Batteries must be recycled. Disposals of potentially damaged batteries present hazards of injury from acid leakage and environmental pollution.

NOTE: When removing a battery, always set it on cardboard, newspaper, or surfaces that cannot be damaged by acid fumes or liquid from the battery.



4.1. Moving Your *EasyTravel Lite* in Freewheel Mode when unfolded:

The Freewheel is used for releasing the brake and moving the *Ezee Travel Lite* without operating the motor. There are two ways to enter the freewheel mode: the switch on the controller cover (Fig. 1/13) and the handle on the motor (Fig. 1A/15).

Switch: before activating, make sure your *Ezee Travel Lite* is switched on. Turn freewheel switch to the "freewheel" position and push or pull the *Ezee Travel Lite* manually.

NOTE: The Freewheel mode can only be activated *after* the battery is in place and the power switch (Fig. 6/1) is turned on. Doing otherwise will cause a fault situation indicated by rapid flashing of the Indicator light (Fig. 6/4). See section 4.2.

Handle: pull the handle up to freewheel mode, push it down for driving mode. This is the mechanical freewheel mode and can be activated without the battery pack.

WARNING: The Freewheel switch should never be engaged on a slope when **Ezee** *Travel Lite* is occupied and without assistance to prevent unintended rolling. If Freewheel is activated on a slope unintentionally, turn your **Ezee Travel Lite** power switch OFF immediately to activate brake.

CAUTION: The seat backrest must remain in the upright position.

4.2. "Walking" your Ezee Travel Lite in the folded position

The folded *Ezee Travel Lite* may be pulled along suitcase style (Fig. 8). The battery may be carried in the utility basket.

4.3. Transporting Your Ezee Travel Lite Quickly And Easily

Your *Ezee Travel Lite* can be folded and transported in most car trunks.

When folding for transportation or storage:

- 1. Disconnect battery pack and place in Utility basket
- 2. Fold the *Ezee Travel Lite*, see folding instructions on page 7
- 3. For storage set the *Ezee Travel Lite* in an upright position (Fig. 9)
- 4. For transportation, place *Ezee Travel Lite* components in car trunk as shown in figure 10.



Figure 8



Figure 9



Figure 10

5. HELPFUL HINTS FOR EVERYDAY USE

Before using your *Ezee Travel Lite* take the following precautions:

Ensure that the front column is locked in the desired driving position.

Ensure that the rear frame is locked open.

Speed Controls

Always operate your *Ezee Travel Lite* at a reasonable speed for both your personal safety and others.

Stopping

Dynamic braking will gradually bring your Ezee Travel Lite to a stop when hand control lever is released.

Ramps and Inclines

Lean forward and carefully manoeuvre your *Ezee Travel Lite* up a ramp or incline. Be aware that not all ramps are constructed according to Government standards. The Government standard for wheelchair ramps is 2.5 cm (1 inch) rise per 30 cm (1 foot).

Lower the speed setting to a slow speed when climbing or descending inclines.

Using an elevator

Backing into an elevator allows you to exit forward. Be sure that the elevator is level with the floor.

Opening Doors

When a door opens towards you, approach at an angle and just off to one side. Adjust the speed knob to a low setting. Grasp the doorknob, reverse and pull the door open. For doors that open away from you, position the front bumper against the door, close to the doorjamb on the doorknob side of the door. After you release the latch, use the bumper to push the door open.

WARNING: Be careful with glass doors to avoid personal injury!

6. GENERAL MAINTENANCE AND INSPECTIONS

Minimum maintenance should prevent unnecessary repairs.

Keep Your Ezee Travel Lite Shining

Wipe the seat clean with a damp cloth. Protect the painted parts with a coat of auto wax.

WARNING: Water or excessive moisture around the controller unit or electrical connections may cause contamination of electrical circuitry, damage and malfunction as well as personal injury. Do not leave your *Ezee Travel Lite* in the rain or in excessively humid locations.

Daily Checks

With the *Ezee Travel Lite* switched off, check that the hand control lever mechanism returns to the rest position when you push and release it. If there is a problem do not use the *Ezee Travel Lite* and contact your authorized dealer.

Weekly Checks

- Throttle test: Pull the throttle to the full speed position and switch ON the Power switch. The Ezee Travel Lite
 should not move and the LED on the panel should flash. If the Ezee Travel Lite does move do not use your Ezee
 Travel. Contact your authorized dealer.
- Automatic brake and drive test: This test should be carried out on a level surface with at least 3' / 1 m. clear space around the Ezee Travel Lite:

 Switch the Ezee Travel Lite on.
 Move the control lever only slightly in the forward direction until you hear a "click" when the brake disengages. The Ezee Travel Lite should start to move forwards slowly.
 Immediately release the throttle. You must be able to hear a "click" when the brake engages back, within a few seconds.
 Repeat the test in the reverse direction.
 If you do not hear the brake operating, or the Ezee Travel Lite does not move slowly in the expected direction do not use your EzeeTravel Lite. Contact your authorized dealer.

Monthly Care

Clean upholstery, plastic and metal parts with a mild surface cleaner.

CAUTION: Keep cleaning solvents away from electrical wires and connectors.

Half-Year Care:

- 1. Apply a lubricant/cleaner (such as WD40 or similar) these locations:
 - Column tilt lock pin and housing.
 - □ Battery pack handle axis and lock pins.
- 2. Lightly apply Vaseline or similar lubricant to battery contact pins and springs on front column.
- 3. Check for tire wear.

Fuse Replacement Instructions

A fuse protects the wiring and battery in the event of a short circuit. If your *Ezee Travel* stops abruptly, check the fuse. If it has 'blown', check for overloads or short circuits. The fuse is located inside the battery case. To replace the fuse, open screws (Fig. 15/1) and remove cover (Fig. 15/2). Remove the fuse (Fig. 15/3) from its socket. If 'blown', replace with the same style and size (automotive "blade" type $-20 \, \text{A}$). There is a spare fuse attached to the battery-pack cover.

WARNING: Do not attempt to repair or service the *Ezee Travel Lite* or any of its components, as this will void your warranty.

SERVICE AND REPAIR: CONTACT YOUR AUTHORIZED DEALER.

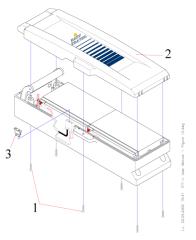


Figure 15

7. EMI – ELECTROMAGNETIC INTERFERENCE

CAUTION: It is important that you read this information regarding the possible effects of electromagnetic interference on your *Ezee Travel*.

Electromagnetic Interference (EMI) From Radio Wave Sources

Motorized scooters may be susceptible to electromagnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones. The interference (from radio wave sources) can cause scooters to release their brakes, come to a sudden stop, or move in an uncontrolled manner. It can also permanently damage a scooter's control system. The intensity of the interfering EM energy can be measured in volts per motor (V/m). Each Motorized scooter can resist EMI up to a certain intensity. This is called its "immunity level". The higher the immunity level, the greater the protection. The FDA has stated that all newly manufactured electric mobility vehicle models should have a resistance of at least 20 V/m, which would provide a reasonable degree of protection from the more common sources of radiated EMI.

Your *Ezee Travel* as shipped, with no further modification, has an immunity level of 20 V/m. This immunity was tested with the inclusion of these accessories: a utility basket with a spare battery-pack and the charger unit carried inside it.

There are a number of sources of relatively intense electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not apparent and exposure is unavoidable. However, we believe that by following the warnings listed below, your risk to EMI will be minimized.

The sources of radiated EMI can be broadly classified into three types:

- 1) **Hand-held portable transceivers** (transmitters-receivers) with the antenna mounted directly on the transmitting unit. Examples include citizens band (CB) radios, "walkie-talkies", security, fire, and police transceivers, cellular telephones, and other personal communication devices.
 - **NOTE: Some cellular telephones and similar devices transmit signals while they are ON, even when not being used;
- 2) **Medium-range mobile transceivers**, such as those used in police cars, fire trucks, ambulances, and taxis. These usually have the antenna mounted on the outside of the vehicle; and
- 3) **Long-range transmitters and transceivers**, such as commercial broadcast transmitters (radio and TV broadcast antenna towers) and amateur (HAM) radios.

NOTE: Other types of hand-held devices, such as cordless phones, laptop computers, AM/FM radios, TV sets, CD players, and cassette players, and small appliances, such as electric shavers and hair dryers, so far as we know, are not likely to cause EMI problems to motorized scooters.

Motorized scooter Electromagnetic Interference (EMI)

Because EM energy rapidly becomes more intense as one moves closer to the transmitting antenna (source), the EM fields from hand-held radio wave sources (transceivers) are of special concern. It is possible to unintentionally bring high levels of EM energy very close to the motorized scooter's control system while using these devices. This can affect the scooter's movement and braking. Therefore, the warnings listed below are recommended to prevent possible interference with the control system of your *Ezee Travel*.

WARNINGS

Electromagnetic interference (EMI) from sources such as radio and TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones can affect motorized scooters. Following the warnings listed below should reduce the chance of unintended brake release or powered wheelchair movement, which could result in serious injury.

- 1) Do not operate hand-held transceivers (transmitters-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such an cellular phones, while your *Ezee Travel* is turned ON;
- 2) Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them;
- 3) If unintended movement or brake release occurs, turn your *Ezee Travel* power switch OFF as soon as it is safe to do so:
- 4) Be aware that adding accessories or components, or modifying your *Ezee Travel*, may make it more susceptible to EMI (Note: There is no easy way to evaluate their effect on the overall immunity of your *Ezee Travel*);
- 5) Report all incidents of unintended movement or brake release to your Authorized *Ezee Travel* dealer or service center, and note whether there is a source of EMI nearby.

8. FAULT FINDING

Hereunder are some types of disorders, which can usually be repaired rather simply. If these following measures are unsuccessful, contact your authorized dealer!

Problem		Check Point
The driving of the wheelchair is too slow or insufficient, the status indicator lamp flashes slowly		Batteries are exhausted and need to be recharged
	1 flash	The battery needs charging or there is a bad connection to the battery. Check the connections to the battery or charge batteries.
	2 flashes	There is a bad connection to the motor. Check all connections between the motor and the controller.
The Ezee Travel Lite does not function, the Battery/Status indicator flashes rapidly. Count the number of flashes:	3 flashes	The motor has a short circuit to a battery connection.
	4 flashes	Freewheel is engaged.
	5 flashes	Not used.
	6 flashes	The controller is being inhibited from driving; this may be because the battery charger is connected.
	7 flashes	A throttle fault is indicated. Make sure that the throttle is in the rest position before switching on the scooter
	8 flashes	A controller fault is indicated. Make sure the controller connections are secure.
	9 flashes	The parking brakes have a bad connection. Check the parking brake and motor connections. Make sure the controller connections are secure.
	10 flashes	An excessive voltage has been applied to the controller. This is usually caused by a poor battery connection. Check the battery packs and controller connections.

9. DISPOSAL AND RECYCLING

- ☐ The packing material must be separated to plastic and paper/cardboard components and submitted to authorized recycling locations.
- The *Ezee Travel Lite* device consists of electronic components, cables, plastic parts, steel and aluminium frame and adapter parts. Do not discard of any components to normal garbage facilities. When *Ezee Travel Lite* is no longer operational, it is to be dismantled and separated into above material groups and submitted to authorized recycling facilities.

10. TECHNICAL DATA

Speed	Continuously adjustable, up to 6 km/h (3.7 mph)
Power Supply	24 V from Two rechargeable maintenance-
	free Sealed Lead/acid batteries
	Standard size: 12V – 7.2 Ah
	Jumbo size: 12V-12 Ah
Charging time	7 – 10 hours
Max. admissible slope	6 ° (10 %)
Max. negotiable curb height	5 cm / 2"
Range approx.*	Standard battery 5 – 6.5 km / 3 - 4 Miles
	Jumbo battery 8 – 10 km / 5 – 6 Miles
Tires	Flat-free PU 200 x 50
Dimensions: (LxWxH)	99 x 55.8 x 86.4 cm / 39" x 22" x 34"
Weights:	
Battery Pack	8.95 kg / 19.75 lb
-	
Total weight (without battery)	27.9 kg / 61.5 lb
Maximum users weight	115 Kg. / 250 Lb.

^{*}The actual range depends on the operation conditions and the load

WARRANTY

The warranty period for the *EzeeTravel Lite* is twelve months and covers faulty materials and workmanship (consumables not covered: tyres, upholstery, lamp bulbs, plastic coverings and batteries). Worn parts damaged as a result of excessive loading, improper handling, intentional damage or unauthorized maintenance or modification are not covered by the warranty.

For safety and for warranty assurance reasons, any modifications and repair of the *Ezee Travel Lite* or its components must be performed exclusively by authorized personnel and exclusively with original spare parts.

The *Ezee Travel Lite* and its accessories have been designed, manufactured and tested in accordance with the specification of the following:

DIRECTIVE: Medical devices 93/42 EEC



