Safety Precautions

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.

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Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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Straight Cane (non-folding): CH2048

1

****BEFORE USING THE CANE, READ INSTRUCTION MANUAL CAREFULLY TO BECOME FAMILIAR WITH THE PRODUCT.**

Weight Capacity 250lbs (113kg)

Please note two AAA 1.5V batteries are not included.

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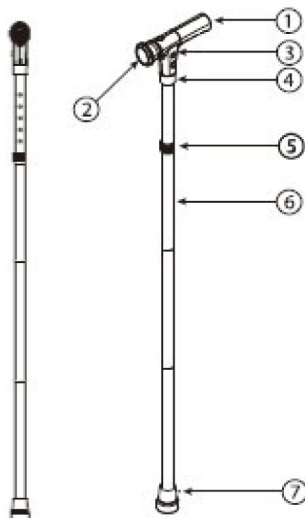
COMPONENTS

1. Handgrip
2. LED light
3. Buzzer
4. Warning light
5. Anti-locking Nut
6. Tube
7. Tip

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2

INTENDED USE

A cane provides support to the user during the task of walking. It is particularly useful for individuals who have limited balance or pain when walking. Canes can be used both indoor and outdoor, however one should always be aware of rough, wet, slippery and/or uneven surfaces. The cane is not suitable for mountain climbing or on rough underground (large stones).

Individuals who have concerns of weak/painful arm or wrist function and/or visual impairments for example should reconsider the use of a cane and consult a professional for advice before using. Using a cane improperly could worsen pre-existing conditions.

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3

WALKING

Follow the advice of your doctor or alternate professional (physiotherapist, occupational therapist for example). If you do not have professional advice, you may consider the following guidelines:

1. Walk upright with the cane next to your body
2. When you begin to walk do not extend the cane further than your front foot
3. The Mayo clinic says, for balance use on either side and when one side is affected, the opposite side is often recommended.

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4

WALKING UP STAIRS

1. When there is a handrail available, you should use it as support on your 'good' side and continue to use your cane on the affected side.
2. Shift your weight on the handrail and place the cane on the first step. Place your unaffected leg on the first step and then follow with your affected leg.

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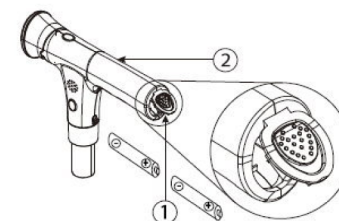
5

PLACING/REMOVING BATTERIES

1. Open the flap on the backside of the handgrip by moving the locking mechanism upwards
2. Insert the batteries in the direction shown in the figure below
3. Close the flap on the backside of the handgrip by moving the locking mechanism downwards

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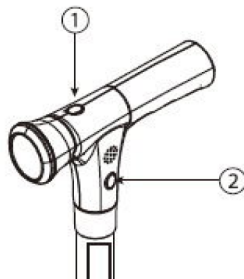
6

OPERATING LED and WARNING LIGHT

1. Press once on the button to activate the LED light
2. Press again, or twice, on the button, the LED and warning light will be lit at the same time
3. Press again, or three times, on the button, the LED light and warning light goes out

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7

OPERATING THE BUZZER

1. Press the panic button, the buzzer should be activated
2. Press the panic button again to turn the buzzer off

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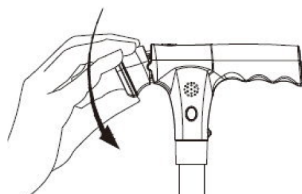
8

ANGLE ADJUSTMENT (LED LIGHT)

1. Grasp the LED light with your hand
2. Move it up and down until the desired angle is achieved (4 positions: 0, 15, -30, and -45 degrees)

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9

HOW TO ADJUST CANE HEIGHT

1. Loosen the adjustment locking ring
2. Press in the spring pin
3. Pull/push the lower tube until the desired height is achieved
4. Ensure the spring pin pops all the way out at the desired level
5. Tighten the adjustment locking ring

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PROPER CANE HEIGHT

For the most accurate and comfortable fit, ensure you wear your everyday shoes and allow someone else to make the adjustment, when choosing the correct cane height.

1. Stand up tall and firm on a flat surface and let your arms hang down naturally by your side.
2. Adjust cane so the top part of the handle touches inside of the wrist where it meets the palm of your hand - Approximately where you would wear your watch.

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CANE HEIGHT RANGE – STRAIGHT CANE (non-folding)

30" - 39"