LOCKING THE CASTERS - VERROUILLER LES ROULETTES

Push down on the locking lever to apply the brakes. To release the brakes, unlock the lever by lifting it up again.

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Safety Precautions

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened

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Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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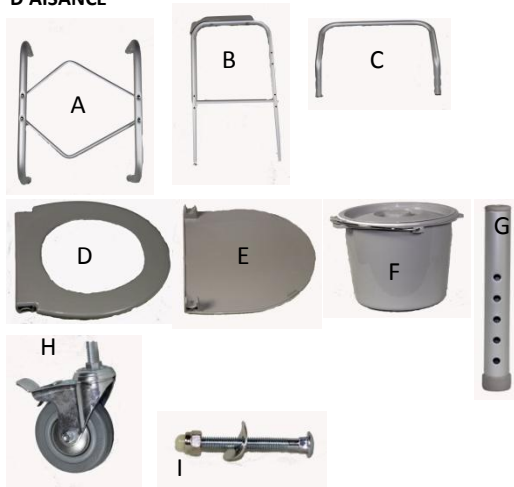
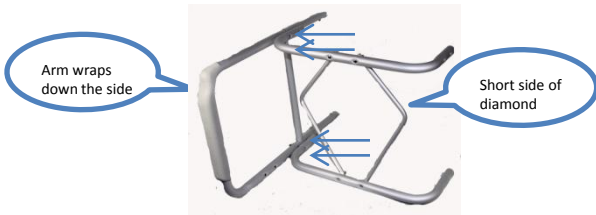
Using 4 screw, nut & bolt combinations to attach the arm support to the seat frame. Repeat on the other side
 Note: Use the bottom set of holes on the arm support
 Note: The padded arm wraps down one side, this is the front and should be facing the same direction as the short end of the diamond shape on the seat frame.

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4

ATTACH THE BACK - FIXER LE DOSSIER

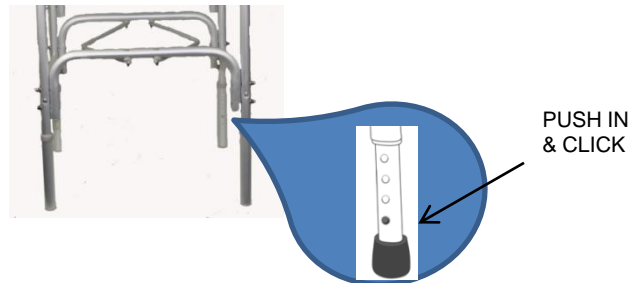
Use 4 screw, nut & bolt combinations to attach the back to the arm supports.
 Note: the back angles outward

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Slide on the adjustable legs and push the button in to adjust the commode height. Repeat on all 4 posts.

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6

ATTACHING THE CASTERS - FIXER LES ROULETTES

Flip the commode upside down and screw on all 4 casters into the adjustable legs.
 Note: You will have to use your 16mm wrench and turn the nut to the right.

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The toilet seat and lid simply snap on to the back seat frame bar.
 Drop the pail (if necessary) into the seat frame.

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