
**Chair and Overbed table  
Fix and tilt section (Split tilt)  
Adjustable in Height and Width (28.5-38.5")  
PRO-TABLE-ADJ**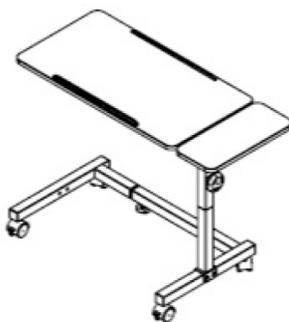

#### Limited Warranty

Pro-Aide Medic products have a limited warranty for one year from the date of purchase against faulty workmanship or materials. In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur. The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as casters, table top and any other parts that are subject to normal wear and need periodic replacement. The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instruction. If you have a question about your Pro-Aide Medic device or this warranty, please contact an authorized Pro-Aide Medic dealer.

#### Assembly (Please refer to the diagram included with this manual)

*Tools needed: hex wrench*

**WARNING:** Keep empty bags used for packaging out of the reach of children.

Step:

1. Insert the hardware (E, F) into the corresponding holes under the table and tighten them. Install the wheels with brakes (F) at the intersection of the 2 tubing

so that they are easily accessible. Putting them in the wrong place will prevent you from applying the brakes.

2. Position the mast (C) on the base (D). It is here that you decide if you want to have the height adjustment mast to the right or left of the U-shaped base. Once decided, please align the holes of the U-Shaped base and the mast. Insert the hardware (H, I) into the holes. Tighten them to solidify the mast to the base of the table.
3. Insert the tilting table top (A) into the tubing. To slide it, you must press the push button to allow the tubing to slide to the end. Make sure the push button secure the tilting table top after sliding in. This button will prevent the table top to get out of the tubing.
4. Insert the fixed section of the table top (B) into the tubing for this purpose. Insert the E clip into the hole to secure in place the fixed section of the table top. Insert the hardware in order to plug the unused holes of the base.
5. Insert the screw into the hole provided for this purpose under the base. It is here that you decide the width of the base.
6. Adjust the height of the table.

If the table is intended for use in a living room with chair, please measure the width of the chair and make sure to widen the base accordingly. The base does not widen more than 38.5"

**WARNING: Ensure that the table is assembled as described, and that all components are securely attached, before use. Make sure the table top is locked in place before use.**










#### USE

Do not lean on the table, it could tilt. The table supports a distributed weight of 30 lbs. Do not exceed this limit. To raise or lower the table top, unscrew the screw provided for this purpose on the mast and bring the table to the desired position. Then tighten the screw to prevent the platform from going up or down. To tilt the table top, press the handle located above the screw on the mast, tilt the table top and release the handle to the desired position. To lock the casters, press the wheel lock button. To unlock them, pull on it.

#### MAINTENANCE

Check the fit and tightness frequently of all nuts and bolts to ensure that the table is securely assembled. To clean: Gently wipe the table with a soft, clean, damp cloth. Then dry with a soft cloth. **Do not use cleansers that may damage the table' finish.**

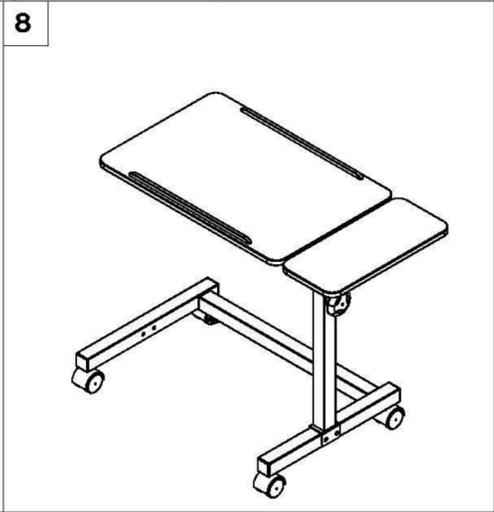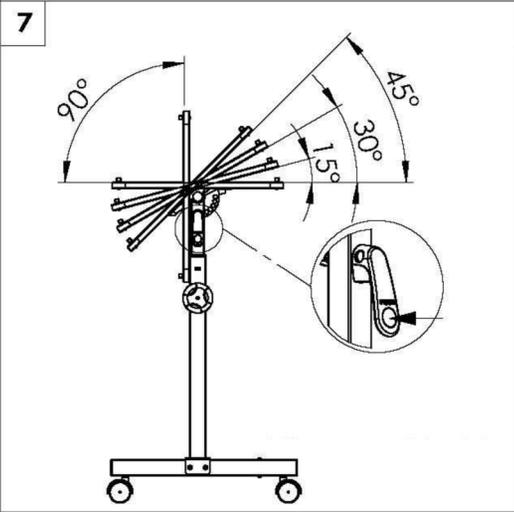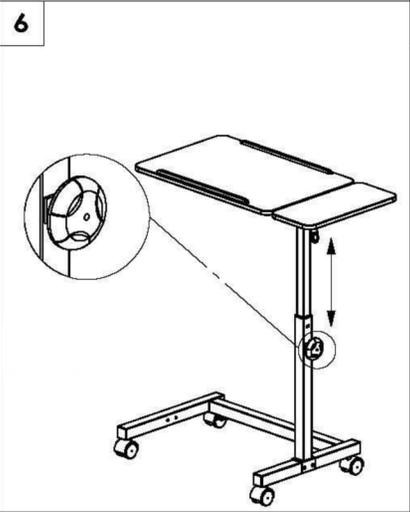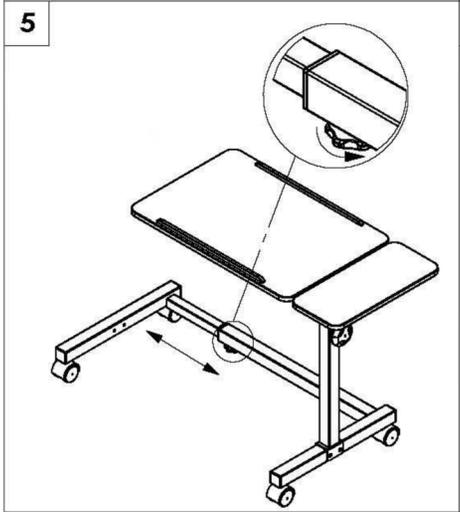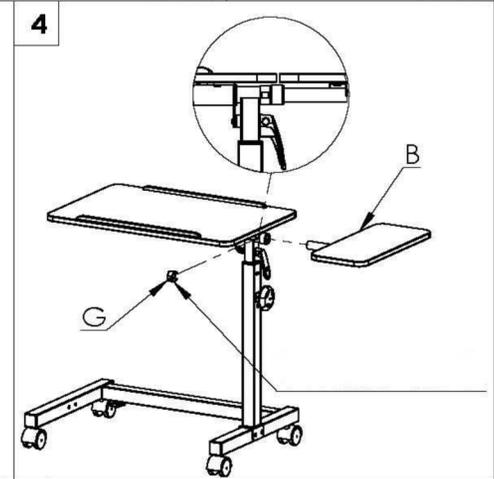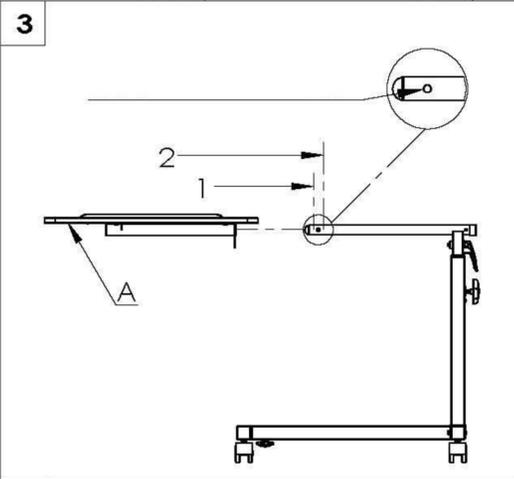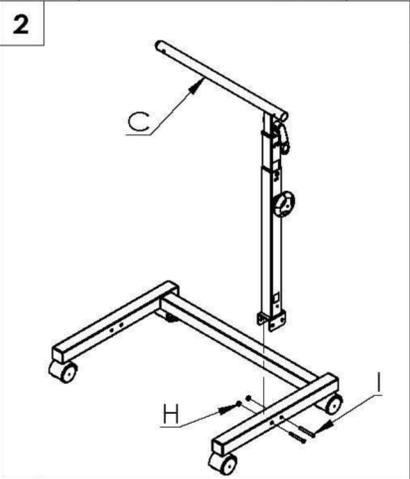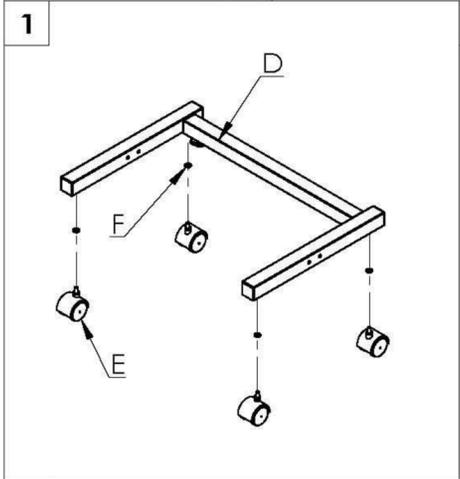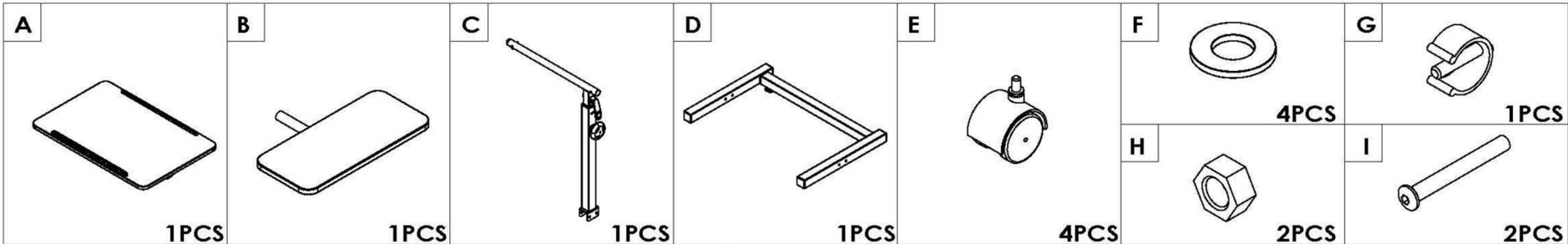Made in Taiwan  
Pro-Aide Medic inc.  
15 Rue Boutet #2  
Quebec, Qc, G2B 2Y6

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